# Tutorial

## RM GE:

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# Gemelli

gen. '22

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The screenshot displays the 'Image Management' software interface with three data tables. The 'Exam' table at the top shows 117 items, with the last row (Exam 25320) selected. The 'Series' table in the middle shows 19 items, with the first row (Series 1) selected. The 'Image' table at the bottom shows 0 items. In each table, a red 'X' icon (representing the 'Delete' function) is visible in the top right corner of the table area, and this icon is only present when one or more rows are selected. A blue arrow points to the 'Delete' icon in the 'Exam' table, and red circles highlight the 'Delete' icons in the 'Series' and 'Image' tables.

Exam	Station name	Name	Date	Description	Modality	Archived	Transferred
25320	MR450W			RM ENCEFALO (S/C M...	MR		
25325	MR450W			RM ADDOME INFERIO...	MR		
25324	MR450W			RM ADDOME SUPERIO...	MR		
25323	MR450W			RM ADDOME INFERIO...	MR		
25322	MR450W			RM ADDOME SUPERIO...	MR		
25321	MR450W			RM ADDOME SUPERIO...	MR		
25320	MR450W			RM PROSTATA (S/C M...	MR		

Series	Type	Images	Description	Modality	Manufacturer	Archived	Transferred
1	PROSP	15	3-plane Loc	MR	GE MEDICAL SYS...		
2	PROSP	15	3-plane Loc	MR	GE MEDICAL SYS...		
3	PROSP	192	Cal Body 36 AA1	MR	GE MEDICAL SYS...		
4	PROSP	33	Sag T2 PROPELLER 3mm	MR	GE MEDICAL SYS...		
5	PROSP	45	Ax T1 Panoramica	MR	GE MEDICAL SYS...		
6	PROSP	45	Ax T2 Panoramica	MR	GE MEDICAL SYS...		
7	PROSP	192	Cal Body 48 AA	MR	GE MEDICAL SYS...		

Image	Date	Image Size

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The screenshot displays three panels in the Image Management software interface:

- Exams Panel:** Shows a list of exams with columns: Exam, Station name, Name, Date, Description, Modality, Archived, and Transferred. A red box highlights the 'Delete' button in the top right corner.
- Series Panel:** Shows a list of series with columns: Series, Type, Images, Description, Modality, Manufacturer, Archived, and Transferred. A blue arrow points to the 'Delete' button in the top right corner, which is also circled in blue.
- Images Panel:** Shows a list of images with columns: Image, Date, and Image Size. A red box highlights the 'Delete' button in the top right corner.

# Image Management

# Exams: 117    Select: [dropdown]    Find: Name [dropdown] [X]

Exam	Station name	Name	Date	Description	Modality	Archived	Transferred
25436	MR450W			RM ENCEFALO (S/C M...	MR		
25435	MR450W			RM ENCEFALO (S/C M...	MR	▲	
25434	MR450W			RM ADDOME INFERIO...	MR		
25433	MR450W			RM ADDOME INFERIO...	MR		
25432	MR450W			RM ADDOME INFERIO...	MR		
25431	MR450W			RM ADDOME SUPERIO...	MR		
25430	MR450W			RM PROSTATA (S/C M...	MR		


# Series: 13    Select: [dropdown] [X]

Series	Type	Images	Description	Modality	Manufacturer	Archived	Transferred to
4	PROSP	36	Ax T2 FSE	MR	GE MEDICAL SYS...	▲	
5	PROSP	36	Ax T2 FLAIR	MR	GE MEDICAL SYS...	▲	
6	PROSP	72	Ax DWI b1000 Tetra	MR	GE MEDICAL SYS...	▲	
7	PROSP	92	AX 3D SWAN	MR	GE MEDICAL SYS...	▲	
8	PROSP	36	Ax T1 FSE	MR	GE MEDICAL SYS...	▲	
9	PROSP	48	Cor T2 FSE	MR	GE MEDICAL SYS...	▲	
650	DIFFUSION	36	ADC (10 <sup>-6</sup> mm <sup>2</sup> /s):Jan...	MR	GE MEDICAL SYS...	▲	

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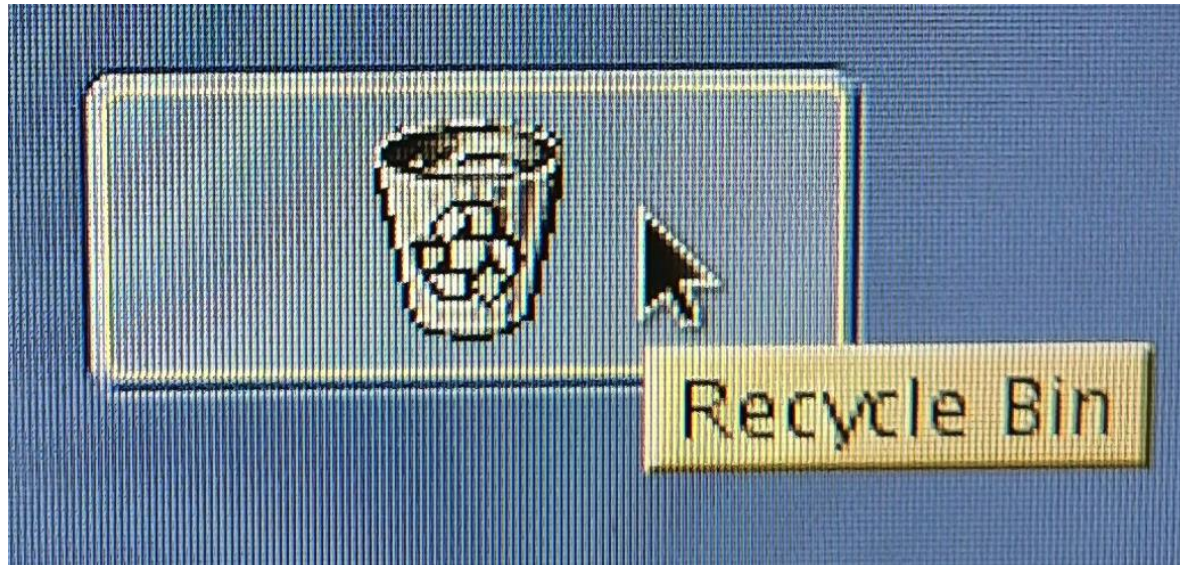
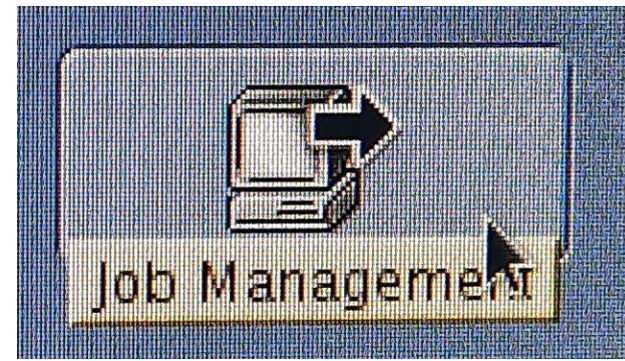
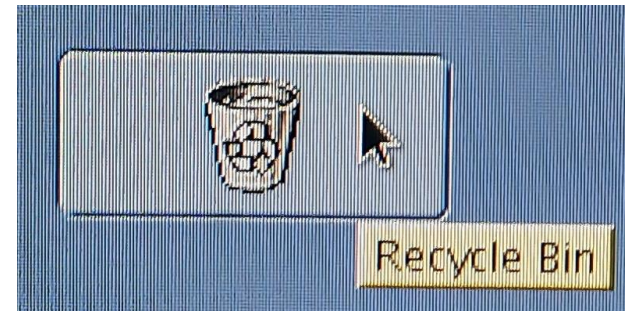
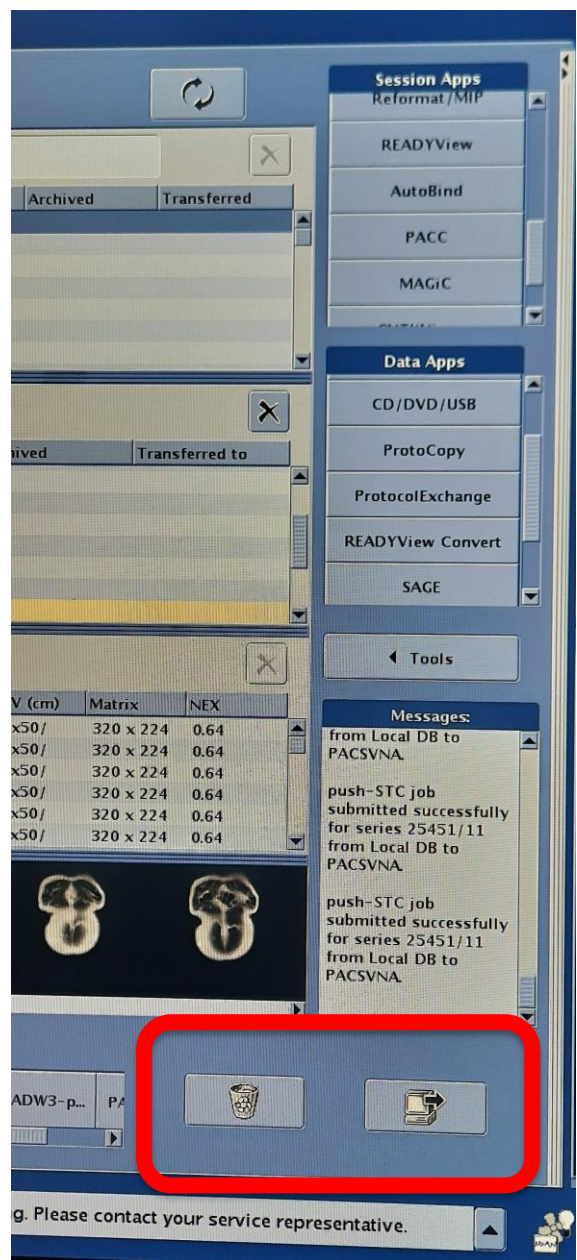
The screenshot shows the 'Job Management' window with the following sections:

- Job Queue:** A table with columns: E/S/I, Job Type, Status, Progress, Submitted On, Source, Destination. It is currently empty.
- Completed Jobs:** A table with columns: E/S/I, Job Type, Completion Status, Submitted On, Source, Destination. It contains three rows of completed jobs.
- Buttons:** Pause, Retry/Resume, Delete, Clear, and Close.

E/S/I	Job Type	Status	Progress	Submitted On	Source	Destination

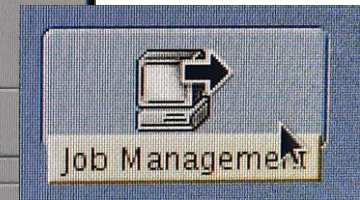E/S/I	Job Type	Completion Status	Submitted On	Source	Destination
25434...	push-S...	All images are stored succes...	21 01 2022	Local DB	PACSVNA
25434...	push-S...	All images are stored succes...	21 01 2022	Local DB	PACSVNA
25434...	pps	Completed.	21 01 2022	Local DB	

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# Series: 19

Select: [dropdown]

Series	Type	Images	Description	Modality	Manufacturer	Archived	Transferred to
--------	------	--------	-------------	----------	--------------	----------	----------------

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Buttons: Restore, Delete, Empty, Clear

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This work is licensed under a Creative Commons Attribution 4.0 International License.