Tutorial

DWI PELVI

Gemelli

set. '22

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MAGIC Synthetic DWI

MAGiC (**MAGnetic resonance image Compilation**) DWI is the commercial name of the method to generate synthetically derived b-values on GE MR scanners, also referred to here as **Synthetic DWI**. MAGiC DWI uses DWI images that have been acquired conventionally with two or more b-values. These acquired DWI images are then processed to synthetically calculate DWI images with other b-values using the calculated Apparent Diffusion Coefficient.

MAGIC Synthetic DWI

MAGiC DWI provides you with the ability to prescribe synthetic b-values when setting up a scan protocol or through post-processing after image acquisition is completed. MAGiC DWI prescribed via the scan protocol generates synthetically derived b-values in-line. The intended use case of this feature is to obtain high b-value DWI when it is otherwise impractical to do so.

MAGIC Synthetic DWI

Details **2** Diffusion Acceleration Advanced

b-values: 10

5 NEX for T2: 2.00

3 Diffusion Direction: ALL

of Diffusion Directions: 3

4 # of T2 Images: 1

Synthetic b-values: 3

#	b-value	NEX
1	1000.0	2.0
2	1000.0	1.0
3	1000.0	1.0
4	1000.0	1.0
5	1000.0	1.0
6	1000.0	1.0

#	Synthetic b-value
1	1000.0
2	1000.0
3	1000.0

Recon All Images Optimize TE: Dual Spin Echo:

MAGIC Synthetic DWI

4. Select # of T2 Images:
enter a value in the text
field.

NOTE! If 0 is entered,
then at least two b-values
must be prescribed to use
synthetic b-value. If 1 is
entered, then at least one
more b-value must be
prescribed to use
synthetic b-value.

MAGIC Synthetic DWI

NOTE! Typically, as the b-value increases, increase the NEX value. Increasing NEX has an added benefit of increasing SNR and reducing artifacts caused by motion, especially for navigated or respiratory triggered acquisitions.

MAGIC Synthetic DWI

The screenshot shows the 'MAGIC Synthetic DWI' software interface with the following parameters and tables:

- 6** # b-values: 10
- NEX for T2: 2.00
- Diffusion Direction: ALL
- # of Diffusion Directions: 1
- # of T2 Images: 1
- 8** # Synthetic b-values: 3

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The screenshot displays the 'Details' tab of an MRI software interface, showing various scan parameters. The parameters are organized into columns: Diffusion, Acceleration, and Advanced. The 'Real Time Field Adjustment' checkbox is highlighted with a red box.

Parameter	Value
# of TE(s) per Scan	1.0
Num Shots	1
TE	linimum
Intensity Correction	NONE
Intensity Filter	None
Save Original	<input type="checkbox"/>
3D Geometry Correction	<input type="checkbox"/>
Real Time Field Adjustment	<input type="checkbox"/>
Real Time Center Frequency	<input type="checkbox"/>
Frequency	128
Phase	128
Bandwidth	250.0
Excitation Mode	Selecti...
Shim	Auto
RF Drive Mode	Quadr...
Phase Correct	On

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FOCUS DWI SERVICE

Ax FOCUS DWI 50-1... **GRx** **5:45** **Details** **Diffusion** **Advanced**

Scan Plane: **Oblique** Freq. Dir: **R/L** # of TE(s) per Scan: **1.0** Frequency: **128**

Freq. FOV: **26.0** TR: **8025.0** Num Shots: **1** Phase: **64**

Phase FOV: **0.50** # Slices: **15** TE: **linimum** Bandwidth: **250.0**

Slice Thickness: **3.00** Auto Prep Time: **145** Excitation Mode: **Focus**

Spacing: **0.3** Recon DL Strength: **Mediu...** Shim: **On**

RF Drive Mode: **CP (Q...**

Phase Correct: **On**

Total # Slices: **15**

Max # Slices: **31**

of Acqs: **1**

Rel. SNR ϕ : **107**

Acq Voxel Size: **2.0x2.0x3.0**

Echo Spacing: **0.8**

Intensity Correction: **NONE**

Calibration In Prescan: **On**

Intensity Filter: **None**

Save Original:

3D Geometry Correction:

Real Time Field Adjustment:

Real Time Center Frequency:

Minimum TE: **62.3**

Maximum TE: **333**

Start **R/L** **A/P** **S/I**

Start **R1.1** **P21.9** **S62.3**

End **R3.5** **P47.0** **S23.7**

Chem SAT: **Special**

Contrast

WB-SAR: 0.16 B₁ RMS: 1.04 μ T Mode: First dB/dt: First

FOCUS DWI SERVICE

Ax FOCUS DWI 50-1... GRx 5:45 Details Diffusion Advanced

Scan Plane: Oblique Freq. Dir: R/L # b-values: 2

Freq. FOV: 26.0 TR: 8025.0 NEX for T2: 0.0

Phase FOV: 0.50 # Slices: 15 Diffusion Direction: ALL

Slice Thickness: 3.00 # of Diffusion Directions: 3

Spacing: 0.3 # of T2 Images: 0

Start R/L A/P S/I Total # Slices: 15 # Synthetic b-values: 1

End R3.5 P47.0 S23.7 Max # Slices: 31

Chem SAT: Special # of Acqs: 1 Rel. SNR(%): 107

Acq Voxel Size: 2.0x2.0x3.0 Echo Spacing: 0.8

Recon All Images: Optimize TE: Dual Spin Echo:

#	b-value	NEX
1	50.0	2.0
2	1000.0	12.0

#	Synthetic b-value
1	1500.0

FOCUS DWI SERVICE

Details Diffusion **Advanced**

b-values: 2

NEX for T2: 0.0

Diffusion Direction: ALL

of Diffusion Directions: 3

of T2 Images: 0

Synthetic b-values: 1

#	b-value	NEX:
1	50.0	2.0
2	1000.0	12.0

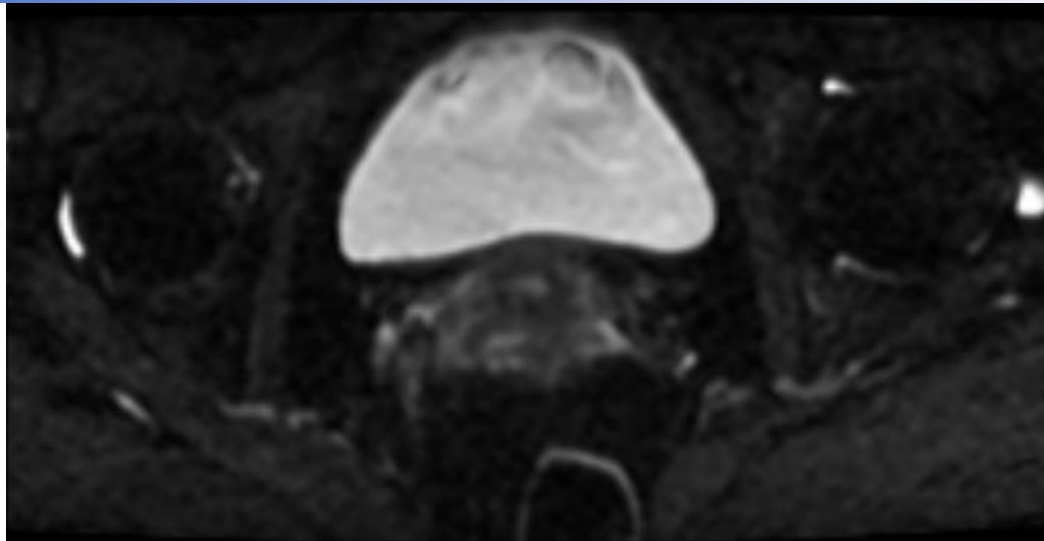
#	Synthetic b-value
1	1500.0

Recon All Images:


Optimize TE: Dual Spin Echo:

FOCUS DWI SERVICE


		User Control Variables		Min	Max
CV0	Ramp Sampling (1= on, 0= off):	<input type="text" value="1.00"/>	0.0	1.0	
CV5	Recon Type (0= Zero Filling, 1= Homodyne):	<input type="text" value="1.00"/>	0.0	1.0	
CV7	Gradient Optimization for Diffusion ALL (1= on, 0= off):	<input type="text" value="1.00"/>	0.0	1.0	
CV9	Shim Volume Mode (0=Default, 1= Breast):	<input type="text" value="0.00"/>	0.0	2.0	
CV39	Receiver Gain (0:Prescan Measured, 1:Predefined):	<input type="text" value="1.00"/>	0.0	1.0	


AX FOCUS DWI
b50-1000
Dir. Freq. R/L

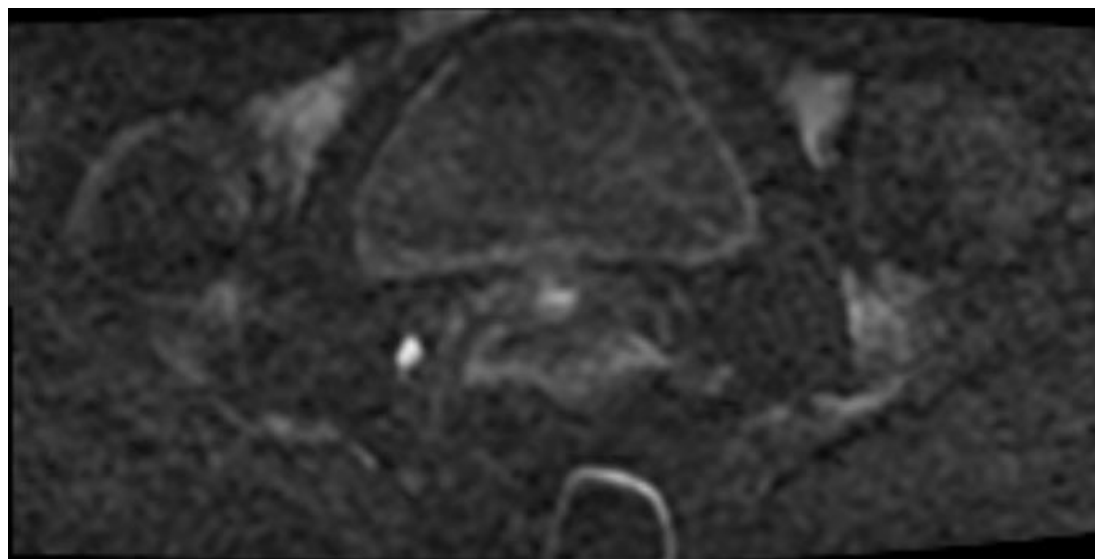AX FOCUS DW_Synthetic b1500

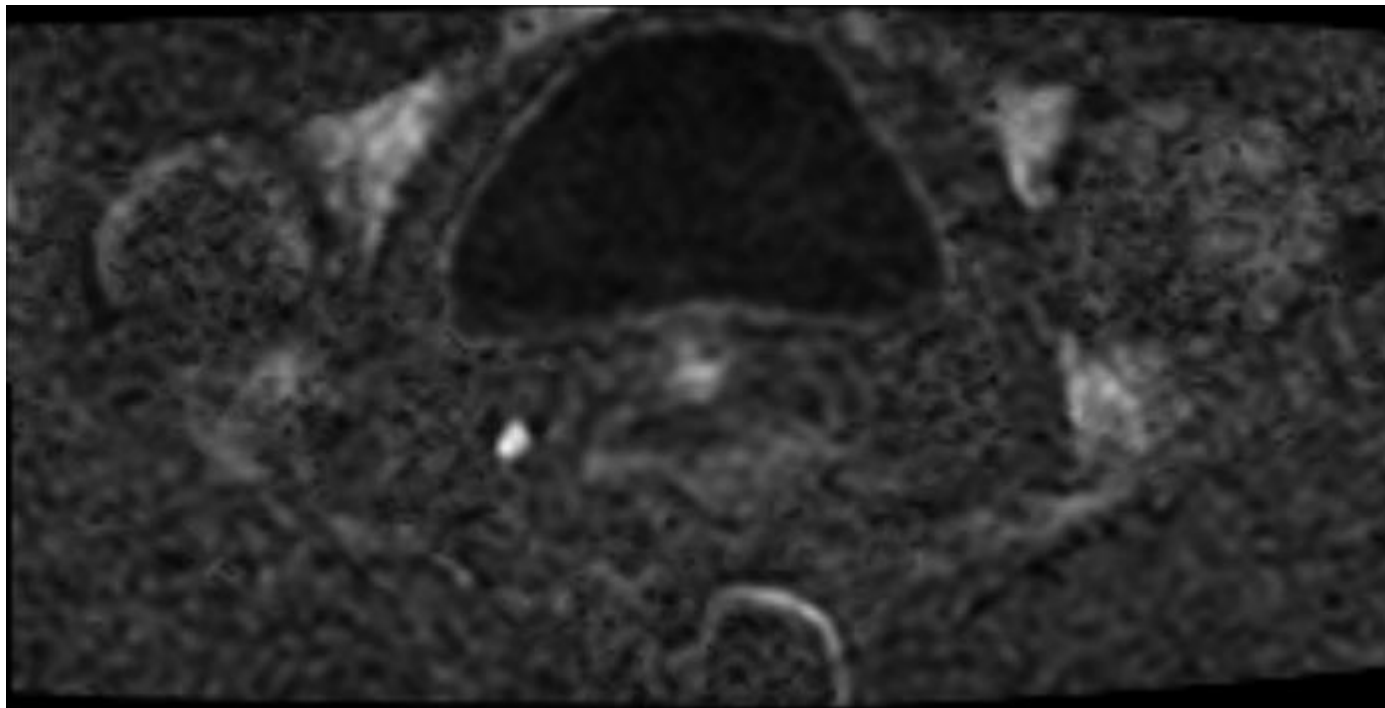
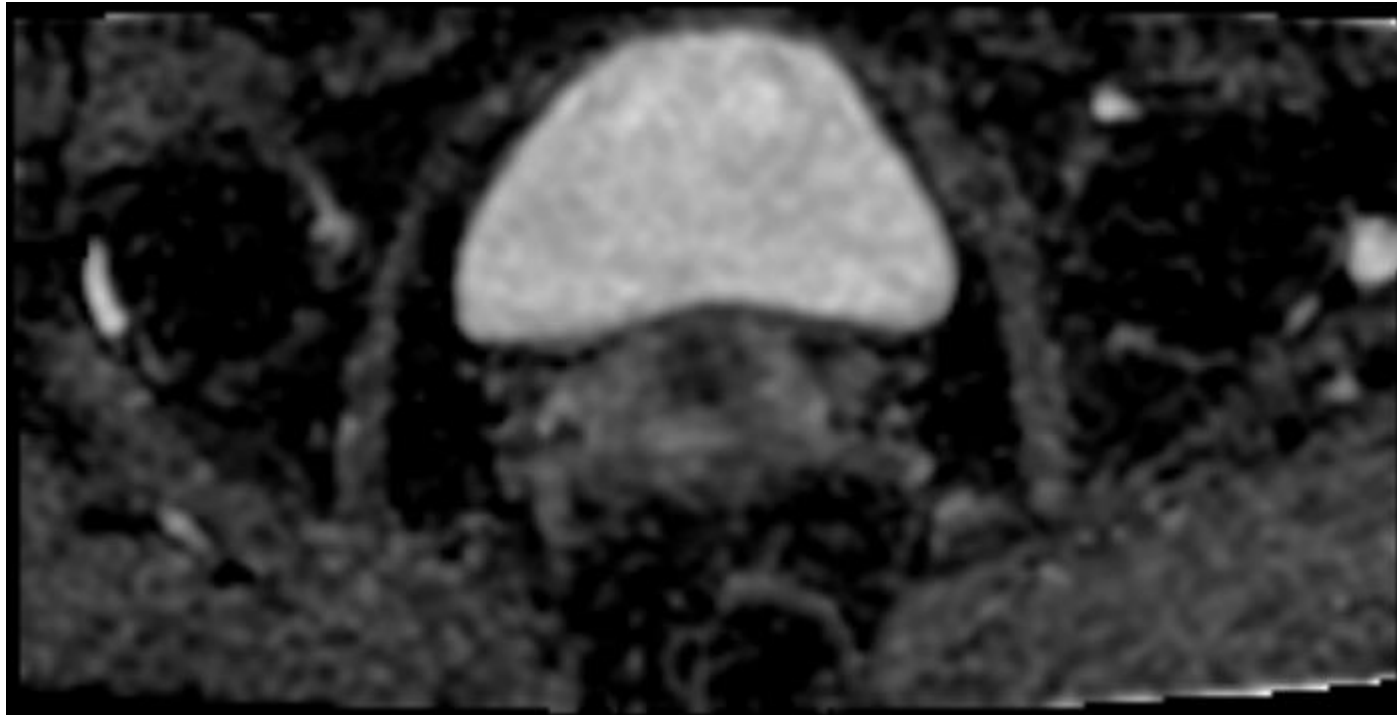
AX FOCUS
ADC ($10^{-6} \text{ mm}^2/\text{s}$)

DWI FOCUS CERVIC

Perché era su 0?

			Min	Max
CV0	Ramp Sampling (1= on, 0= off):	<input type="text" value="1.00"/>	0.0	1.0
CV5	Recon Type (0= Zero Filling, 1= Homodyne):	<input type="text" value="1.00"/>	0.0	1.0
CV7	Gradient Optimization for Diffusion ALL (1= on, 0= off):	<input type="text" value="1.00"/>	0.0	1.0
CV9	Shim Volume Mode (0= Default, 1= Breast):	<input type="text" value="0.00"/>	0.0	2.0
CV39	Receiver Gain (0:Prescan Measured, 1:Predefined):	<input type="text" value="1.00"/>	0.0	1.0

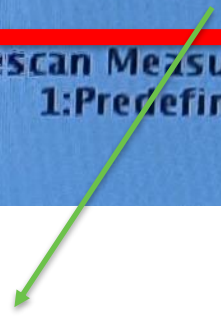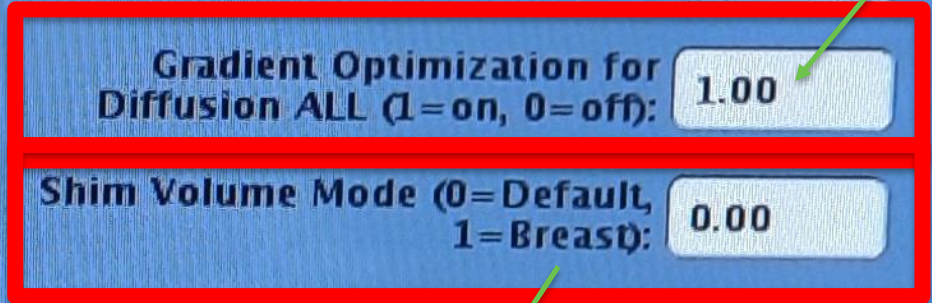GRADIENT OPTIMIZATION FOR DIFFUSION

		Min		Max	
CV0	Ramp Sampling (1= on, 0= off):	1.00	0.0	1.0	
CV5	Recon Type (0= Zero Filling, 1= Homodyne):	1.00	0.0	1.0	
CV7	Gradient Optimization for Diffusion ALL (1= on, 0= off):	1.00	0.0	1.0	
CV9	Spin volume Mode (0= Default, 1= Breast):	0.00	0.0	2.0	
CV39	Receiver Gain (0: Prescan Measured, 1: Predefined):	1.00	0.0	1.0	

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SHIM VOLUME MODE

User Control Variables			Min	Max
CV0	Ramp Sampling (1= on, 0= off):	<input type="text" value="1.00"/>	0.0	1.0
CV5	Recon Type (0= Zero Filling, 1= Homodyne):	<input type="text" value="1.00"/>	0.0	1.0
CV7	Gradient Optimization for Diffusion ALL (1= on, 0= off):	<input type="text" value="1.00"/>	0.0	1.0
CV9	Shim Volume Mode (0= Default, 1= Breast):	<input type="text" value="0.00"/>	0.0	2.0
CV39	Receiver Gain (0:Prescan Measured, 1:Predefined):	<input type="text" value="1.00"/>	0.0	1.0

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
Details Diffusion **Advanced**

User Control Variables

		Min	Max
CV0	Ramp Sampling (1= on, 0= off):	0.0	1.0
CV5	Recon Type (0= Zero Filling, 1= Homodyne):	0.0	1.0
CV7	Gradient Optimization for Diffusion ALL (1= on, 0= off):	0.0	1.0
CV9	Shim Volume Mode (0= Default, 1= Breast):	0.0	2.0
CV39	Receiver Gain (0:Prescan Measured, 1:Predefined):	0.0	1.0

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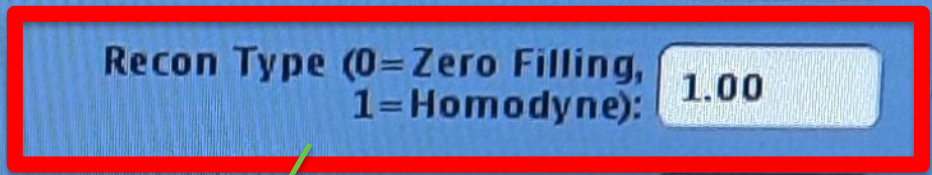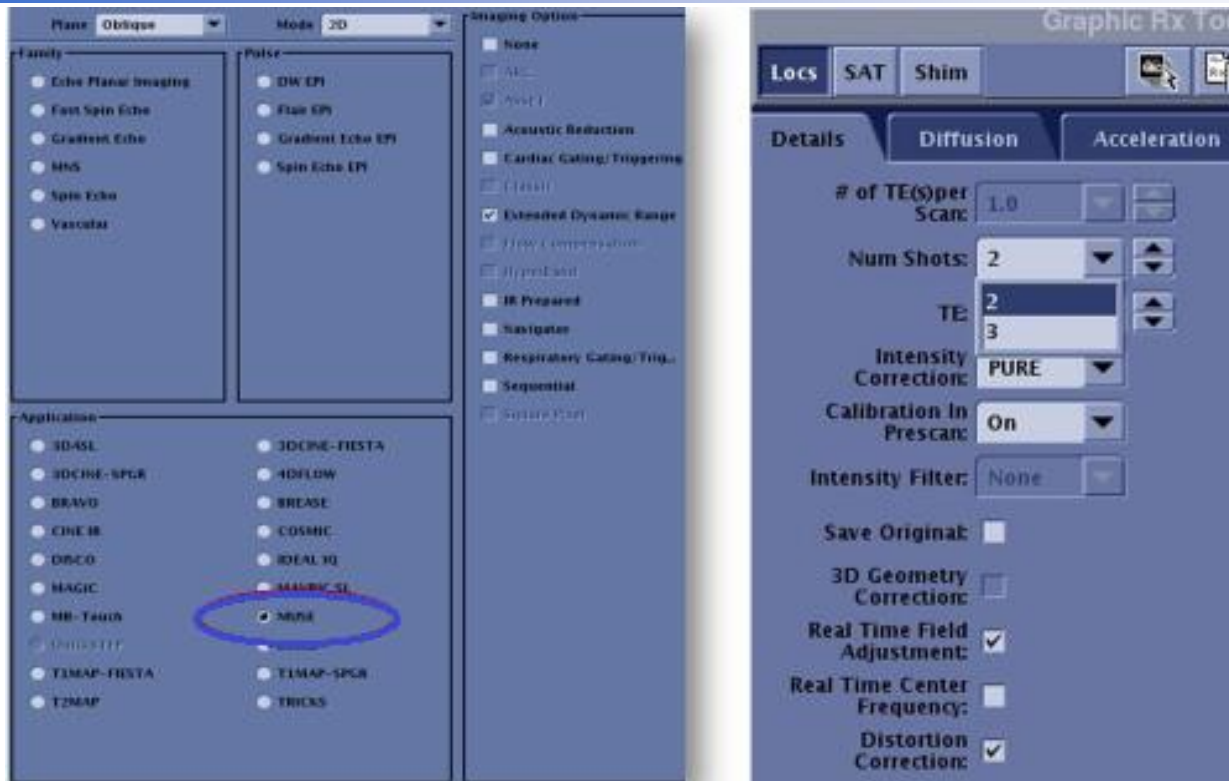
Considerations:

MUSE is compatible with: DWI & DTI
ASSET

Any coil 8 Channels or more

IR Prep (MUSE STIR)

Distortion Correction

MUSE is not compatible with: EPI FLAIR

PROPELLER DWI

HyperBand

Focus DWI

Image reconstruction time is slightly longer than with conventional DWI and DTI

Include all anatomy within the Field of View (Skin to Skin) to avoid image artifacts


The addition of anterior and posterior saturation bands helps to reduce motion and artifacts from fat

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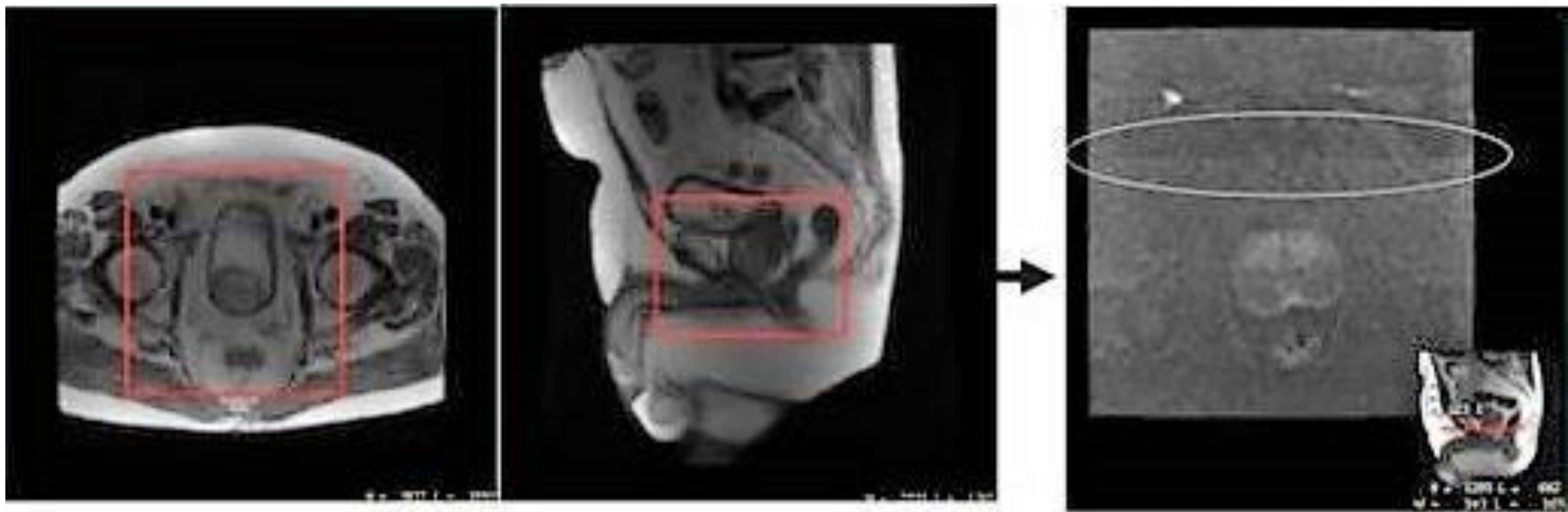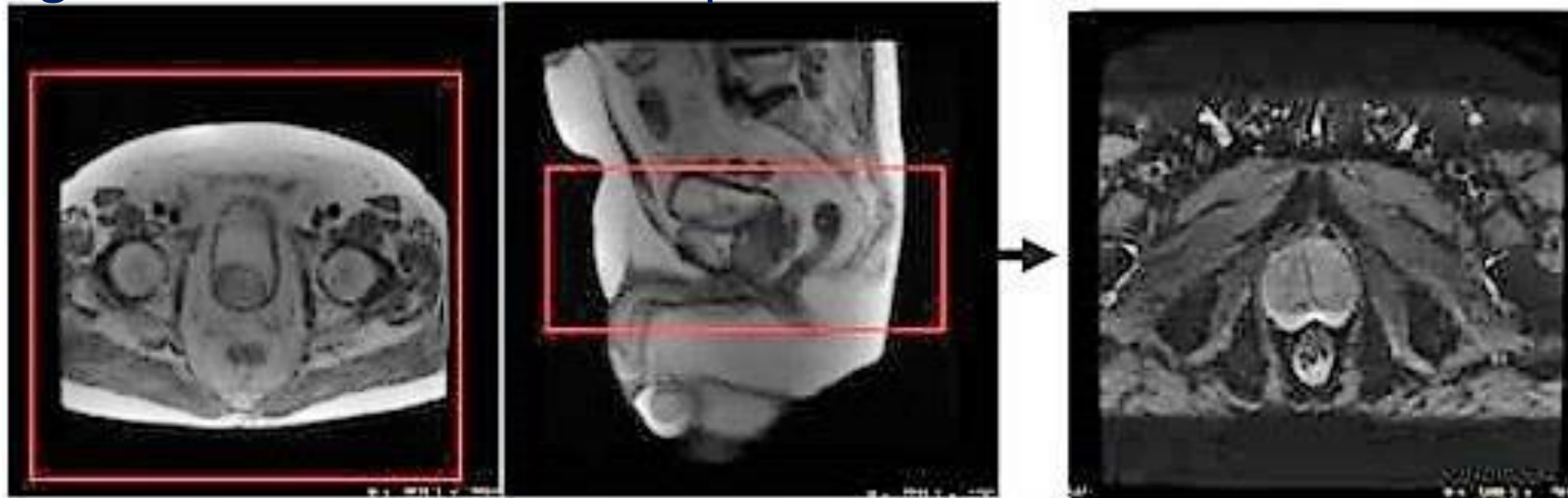
Auto TR Max: 17000

AX MUSE SERVICE

Details Diffusion Acceleration Advanced

b-values: 2

NEX for T2: 0.0

Diffusion Direction: ALL

of Diffusion Directions: 3

of T2 Images: 0

Synthetic b-values: 0

#	b-value	NEX
1	50.0	3.0
2	1000.0	10.0

Recon All Images: Optimize TE: Dual Spin Echo:

Details Diffusion Acceleration **Advanced**

User Control Variables

			Min	Max
CV0	Ramp Sampling (1= on, 0= off):	<input type="text" value="1.00"/>	0.0	1.0
CV5	Recon Type (0= Zero Filling, 1= Homodyne):	<input type="text" value="1.00"/>	0.0	1.0
CV7	Gradient Optimization for Diffusion ALL (1= on, 0= off):	<input type="text" value="1.00"/>	0.0	1.0
CV9	Shim Volume Mode (0= Default, 1= Breast):	<input type="text" value="0.00"/>	0.0	2.0
CV39	Receiver Gain (0: Prescan Measured, 1: Predefined):	<input type="text" value="1.00"/>	0.0	1.0

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The screenshot shows the 'Diffusion' tab of the software interface. The 'Diffusion Direction' is set to '3in1'. A table on the right lists the b-values and their corresponding NEX values:

#	b-value	NEX:
1	50.0	6.0
2	1000.0	16.0

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This work is licensed under a Creative Commons Attribution 4.0 International License.