Tutorial

ARTERIAL SPIN LABELING (ASL)

Gemelli

nov. '22

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This work is licensed under a Creative Commons Attribution 4.0 International License.

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
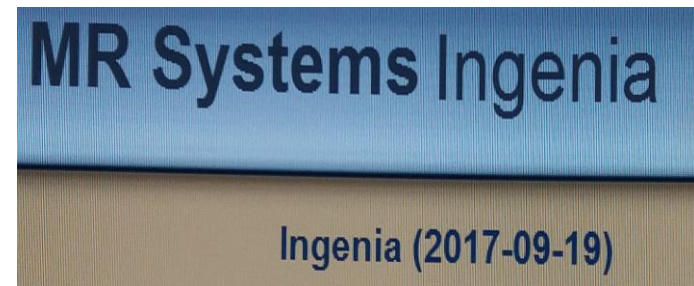
PHILIPS
sense and simplicity

Ingenia 1.5T

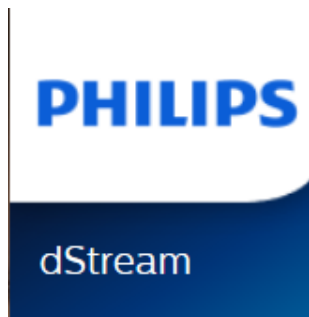Arterial Spin Labeling (ASL) Recommendations

1. Run an EPI test scan and don't run ASL in case of distortions in the neck area.
→ Risk of false positive carotid occlusion.
2. Do not use ASL after contrast media injection.
→ Risk of ASL and CBF maps without signal.
3. Choose the Post-Label Delay (PLD) value according to the patient age.
 1. Pediatric patients [1.0s – 1.5s]
 2. Adult patients [1.5s – 2.0s]
 3. Geriatric/stroke patients [2.5s – 3.0s]
4. Do not oblique the imaging volume; ensure the patient's head is not tilted and the slab coverage begins at the base of the cerebellum.
→ Risk of shading and/or low SNR.

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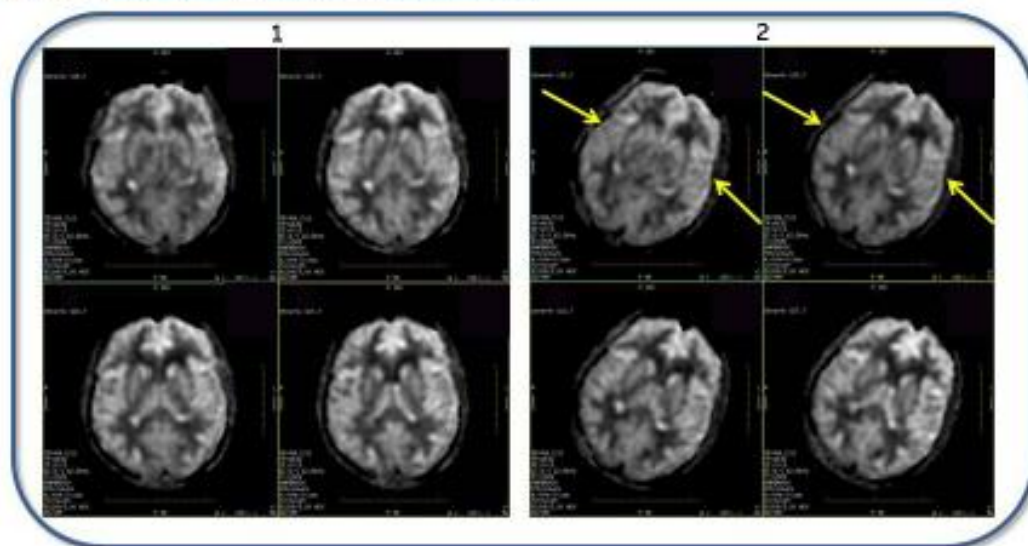
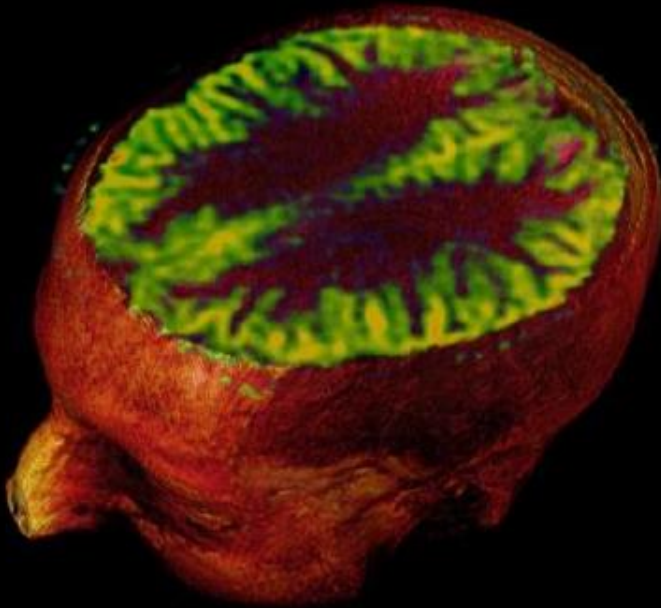
3D ASL Quantitative, Arterial Spin Labeling

What is it?

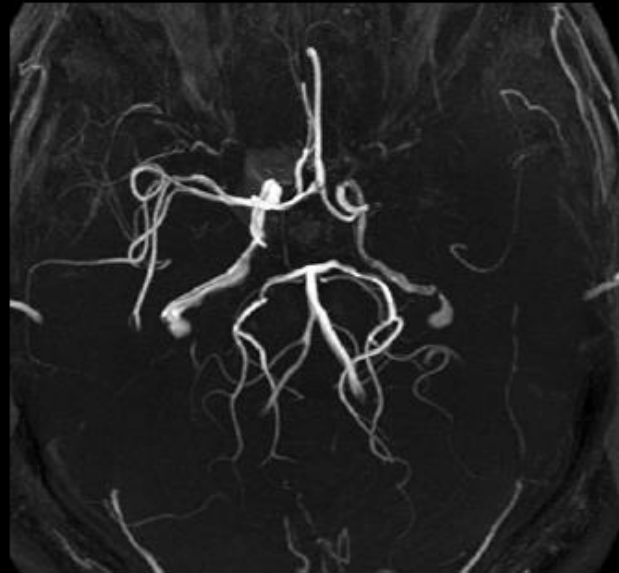
- 3D Arterial Spin Labeling technique that provides quantitative cerebral blood flow volumes
- mL/100g/min

Benefits


- Non-contrast = saves time and money
- Quantitative assessment over time could potentially show progress of therapy


3D ASL

3D TOF

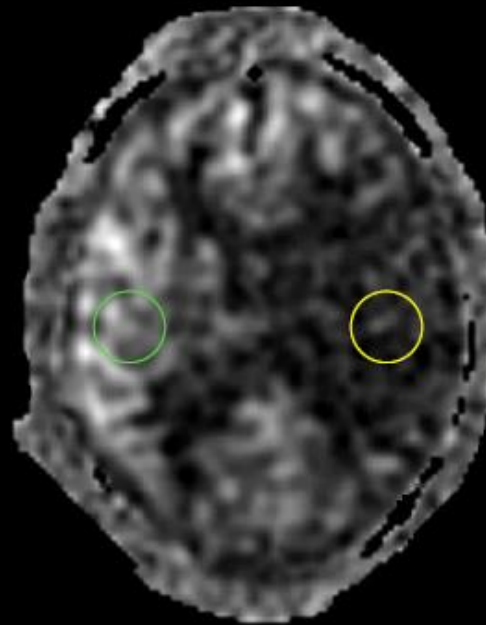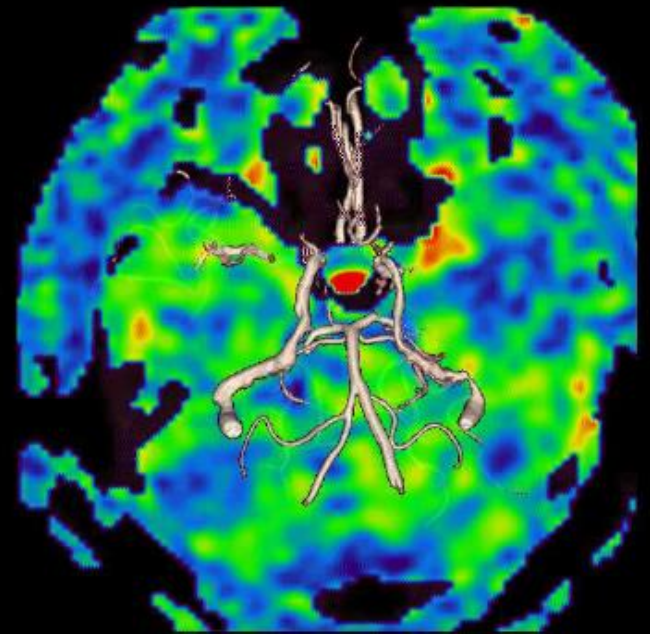3D ASL
19 ml/100g/min (green)
9 ml/100g/min (yellow)


3D ASL/ 3D TOF Fusion


3D ASL

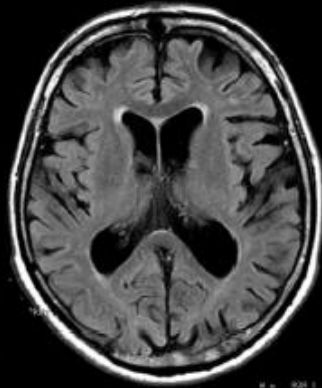
The CBF maps helps to provide a non-invasive way to follow a stroke in a patient with Epilepsy

FLAIR

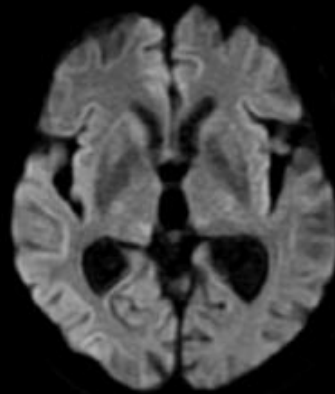DWI

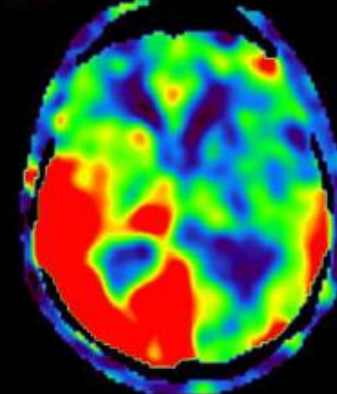3D ASL

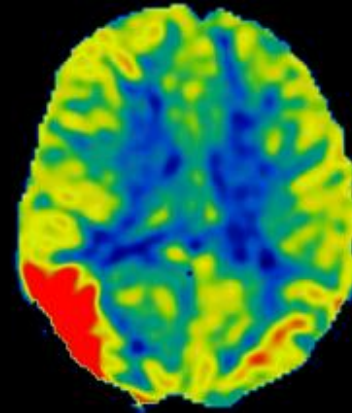
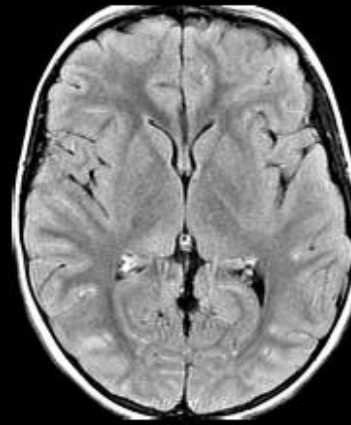
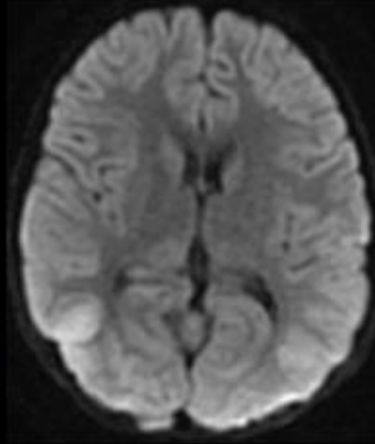
Epileptic Stroke

6 days post
epileptic
stroke




3D ASL

Example of 8 y/o patient with West Nile Encephalitis

Infectious
August 28th

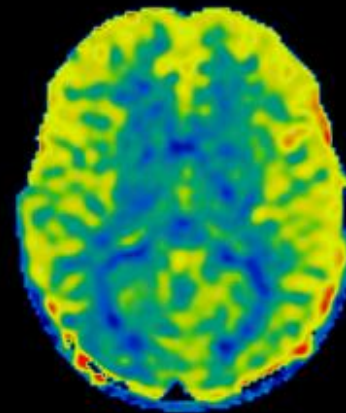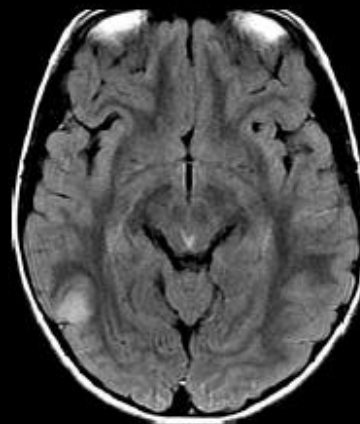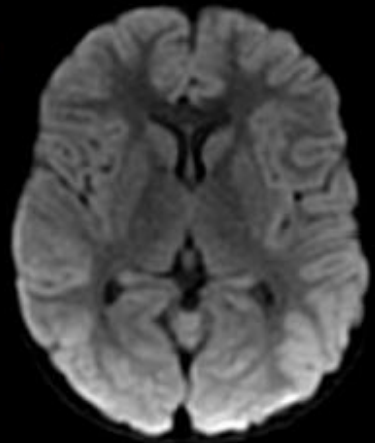The CBF maps helps to visualize the regression of the infection due to treatment

Resolving
October 10th

DWI

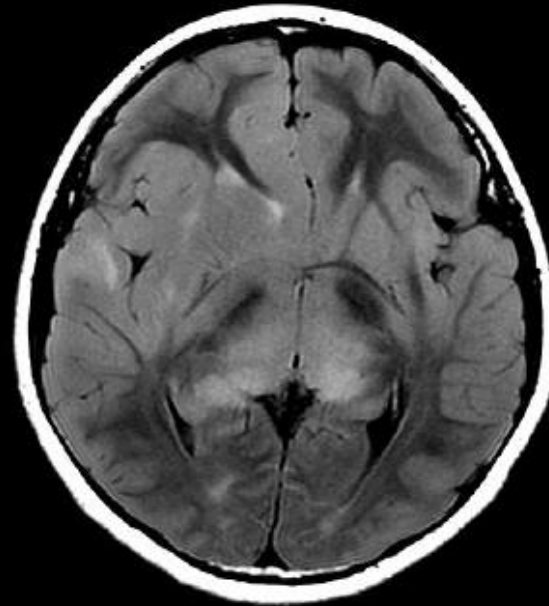
FLAIR

3D ASL


3D ASL

Example of 7 y/o with
Gliomatosis

The CBF map helps to
visualizes tumor
infiltration

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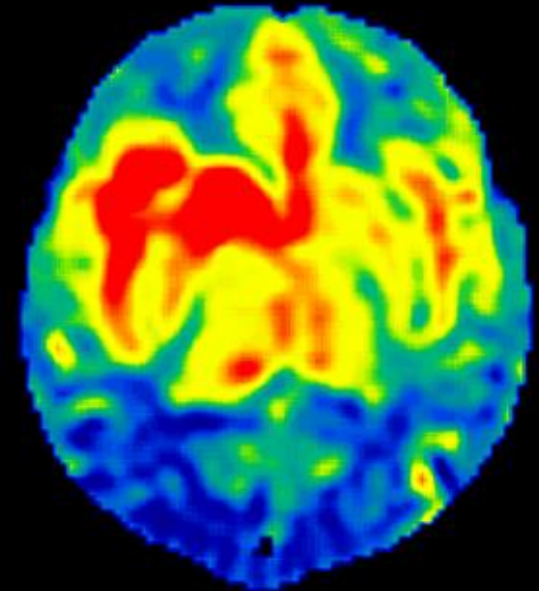pCASL

Pseudo continuous arterial spin labeling

Train of short discrete RF pulses mimic flow-driven adiabatic inversion

Info:

Label distance and post label delay can be adjusted

The sequence exists out of an control and label experiment, which need subtraction to get the perfusion visible.

The pCASL sequence is based on an EPI which might suffer within the source images some EPI artifacts. Those will be subtracted away within post processing package.

The label duration of this sequence is 1800

Tips:

On the source images subtraction needs to be applied. This can be performed within the image algebra package by applying the ASL subtraction.

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

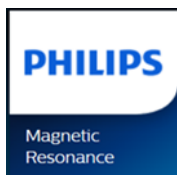
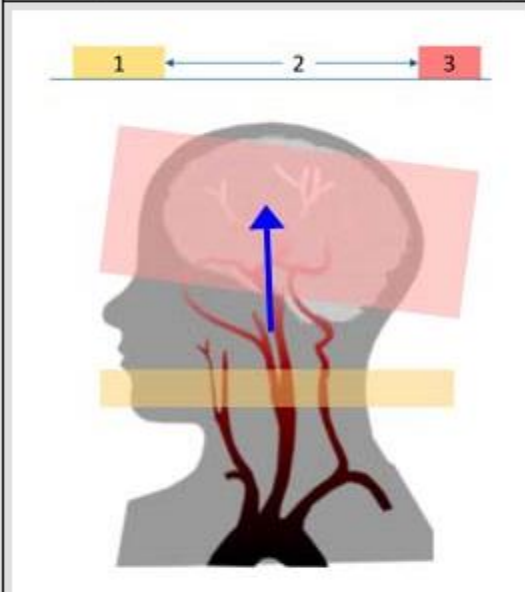
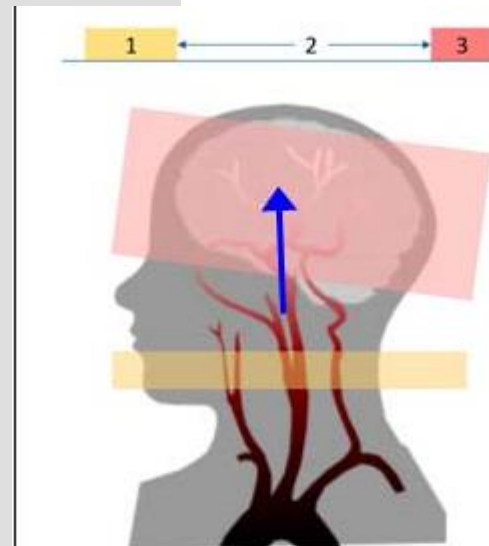
ASL single-phase

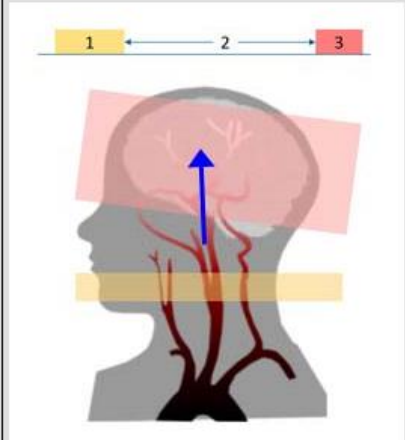
Single-Phase ASL - using the pCASL method

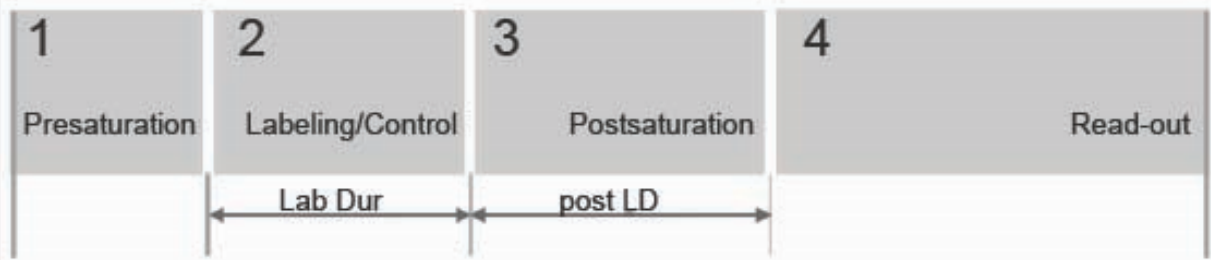1. Labeling and control
2. Post-label delay
3. Image acquisition

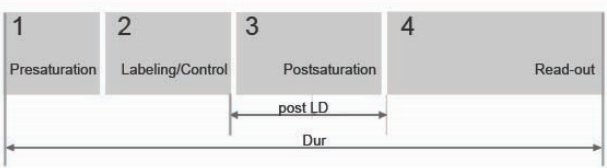1. Pre-saturation
 2. Labeling/Control
 3. Post-saturation
 4. Read-out
- post LD - post label delay
 - Lab Dur - Label duration

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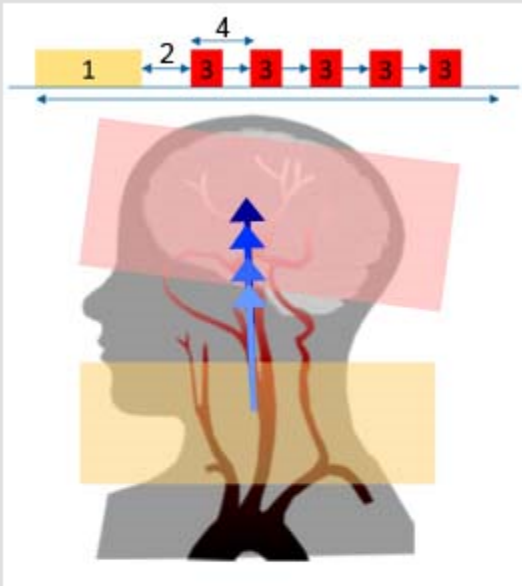
Multi-phase 3D arterial spin labeling brain MRI in assessing cerebral blood perfusion and arterial transit times in children at 3T

<https://pubmed.ncbi.nlm.nih.gov/30439588/>

3D pseudocontinuous arterial spin labeling (pCASL) with a single post-labeling delay time is commonly used to measure cerebral blood flow (CBF). Multi-phase pCASL has been developed to simultaneously estimate **CBF** and **arterial transit time (ATT)**.


ASL multi-phase

Multi-Phase ASL - using the STAR method

1. Labeling and control
2. Label delay
3. Image acquisition
4. Phase interval



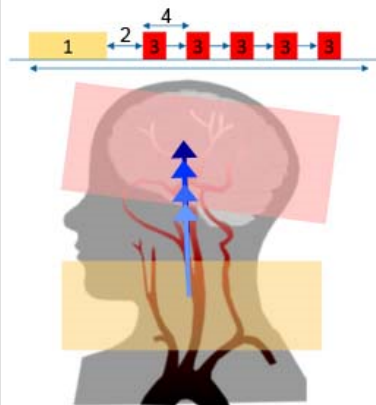
In multiphase ASL, perfusion is measured at multiple time points after labeling: the same area is imaged multiple times after labeling (or control). This results in multiple perfusion images.

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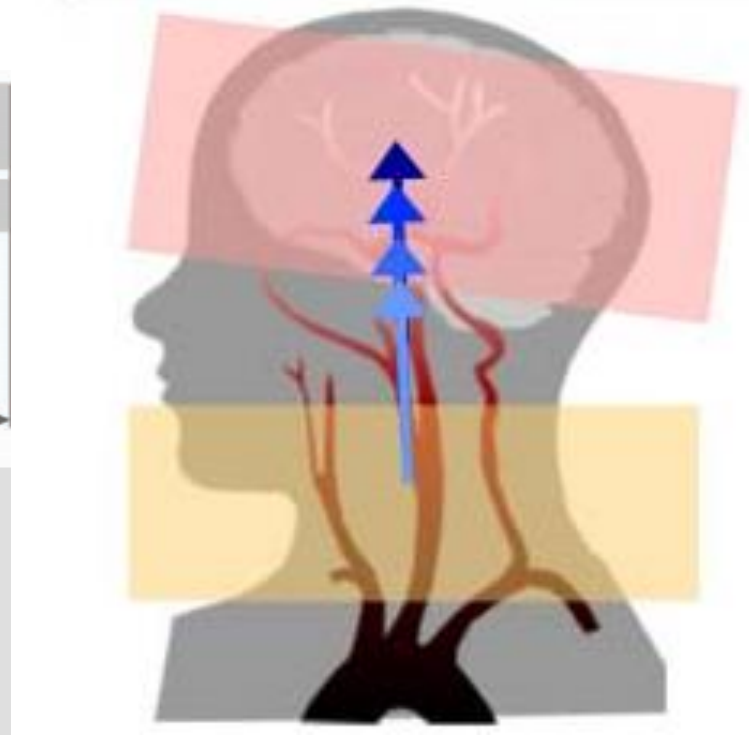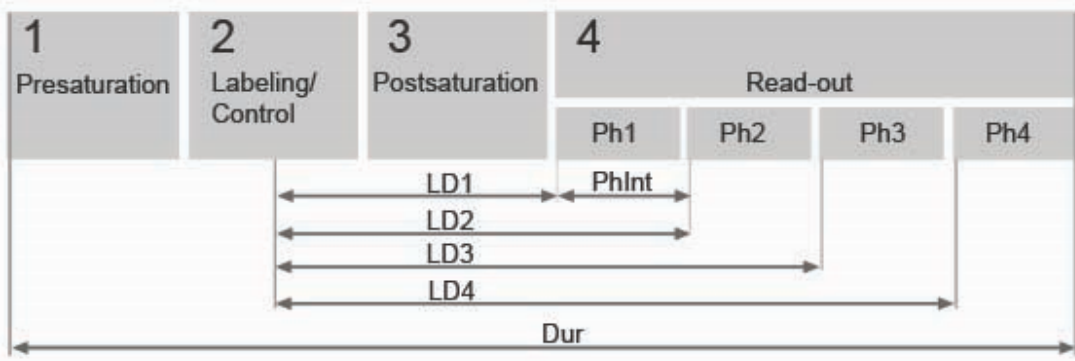
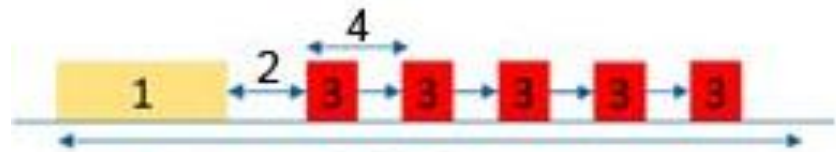
ASL multi-phase

1. Pre-saturation
 2. Labeling/Control
 3. Post-saturation
 4. Read-out
- Ph1, 2, 3, 4 - Phase 1, 2, 3, 4
 - LD1, 2, 3, 4 - Label delay 1, 2, 3, 4
 - where LD1 is displayed as **Label delay 1st** on the Scan Information page

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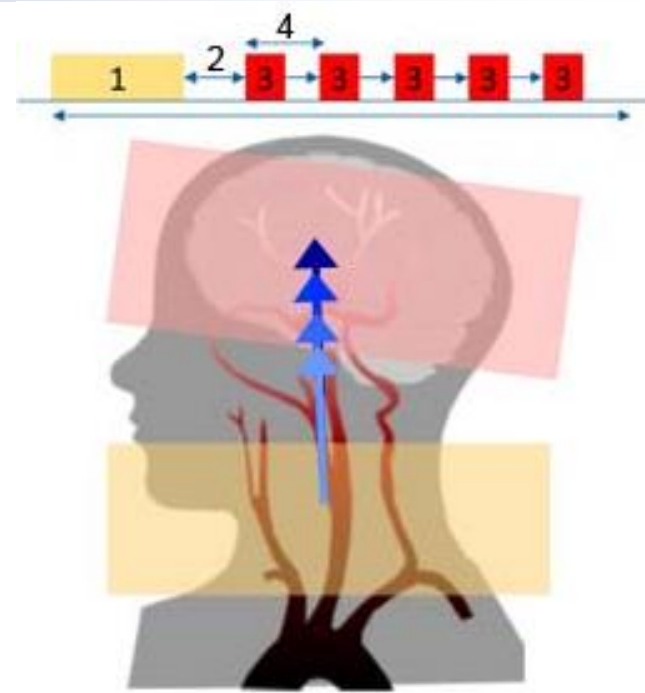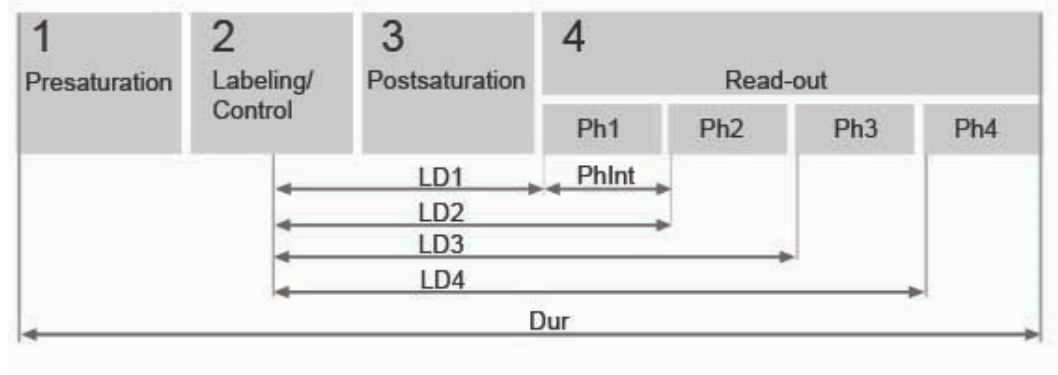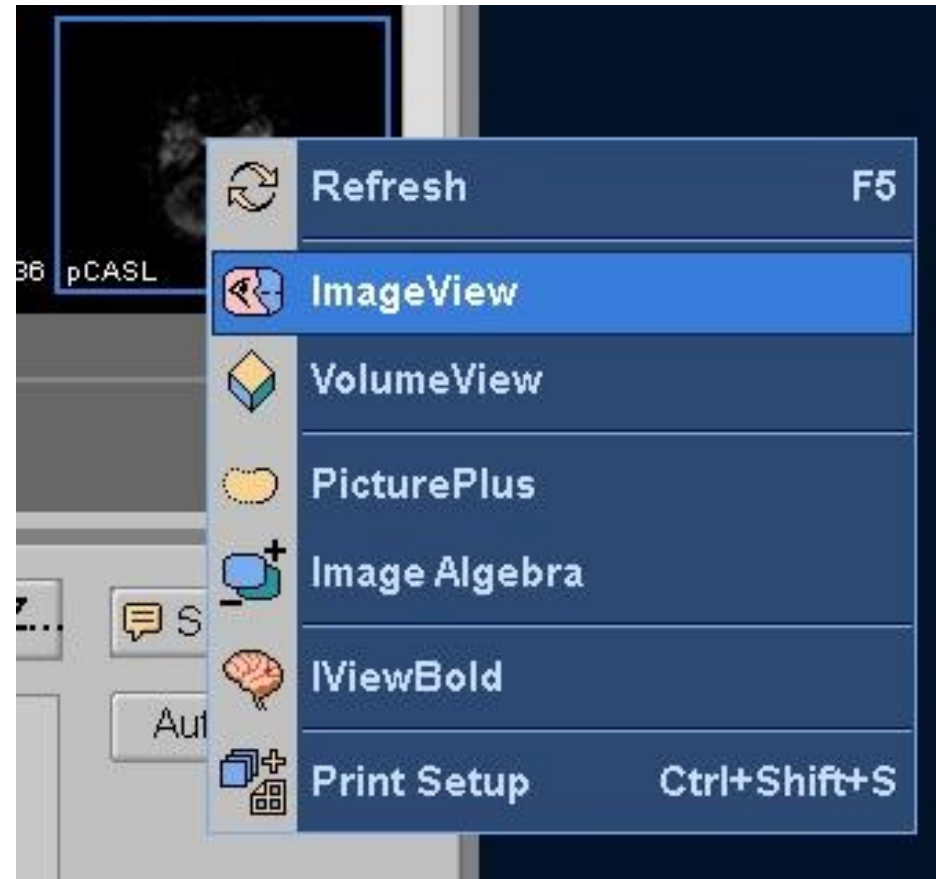
Continuing education program: focus...

Arterial spin labeling (ASL) perfusion: Techniques and clinical use

J.-C. Ferré^{a, b, c, d} ✉, E. Bannier^{b, c, d}, H. Raoult^{a, b, c, d}, G. Mineur^a, B. Carsin-Nicol^a, J.-Y. Gauvrit^{a, b, c, d}

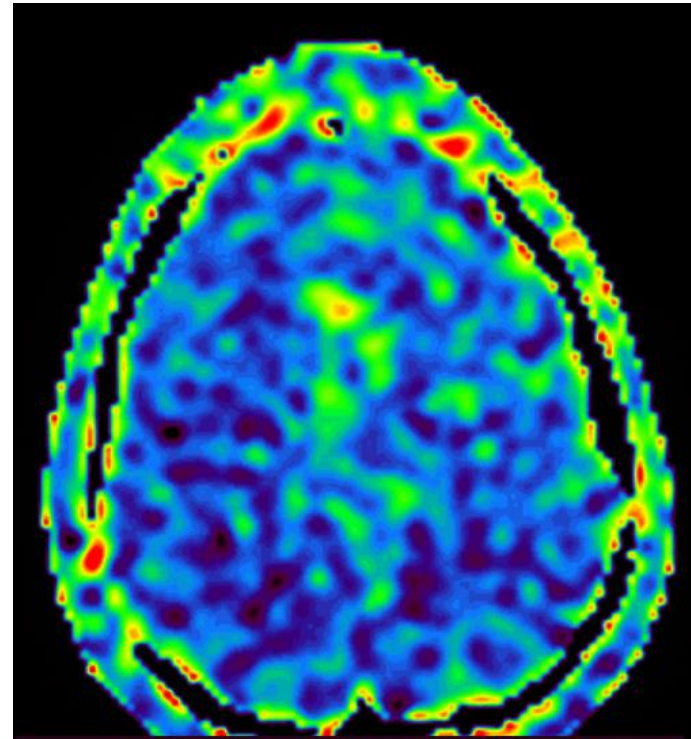
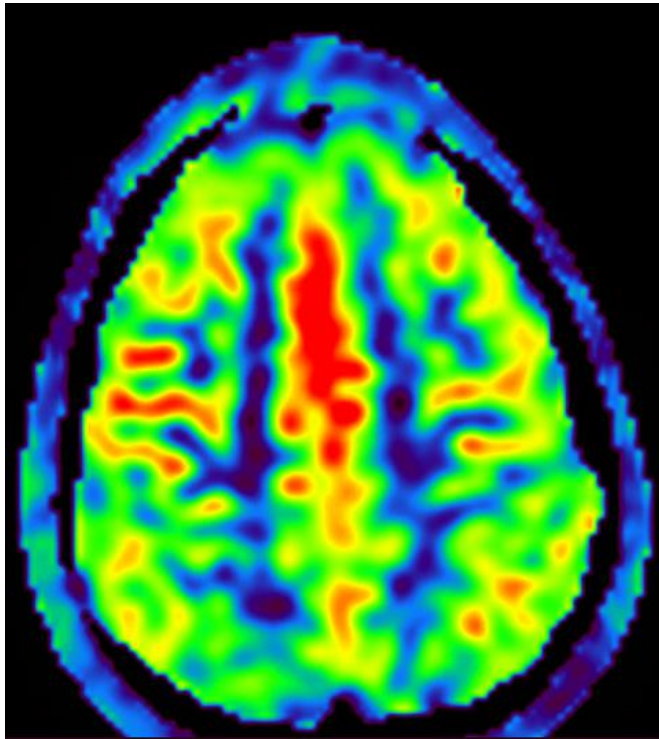
Post-gadolinium arterial spin labeling

Gadolinium contrast media significantly reduce the T1 of the blood. This shortening is such that, after labeling by ASL, it is not possible to detect the minor difference between the labeled and control images. Therefore, it is not possible to carry out an ASL sequence immediately after the injection of gadolinium. A minimum of 12 hours seems necessary with respect to the half-life of gadolinium contrast media.

<https://doi.org/10.1016/j.diii.2013.06.010>



ASL Pre-MdC


ASL Post-MdC


ASL Pre-MdC

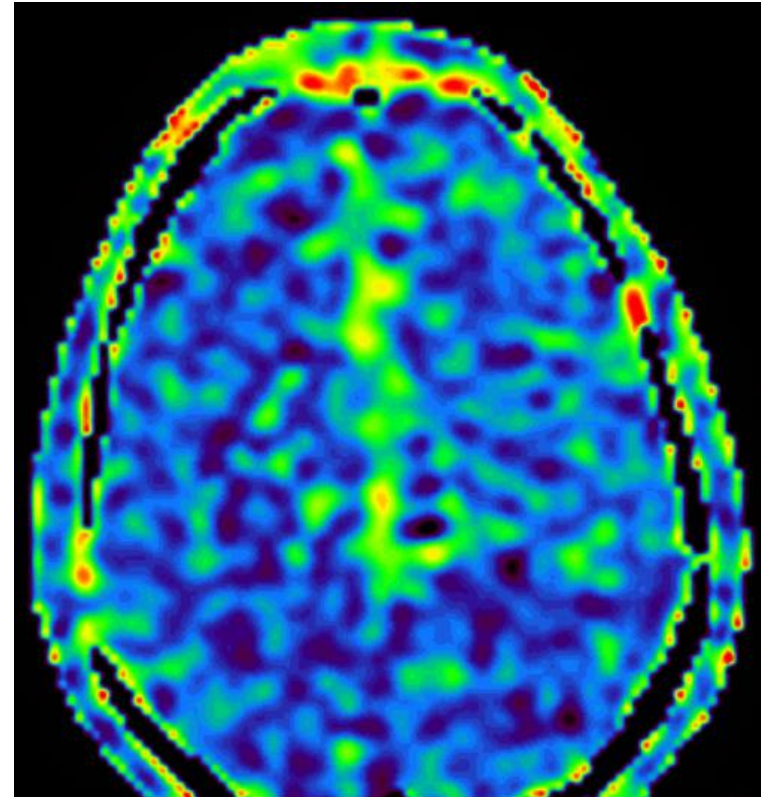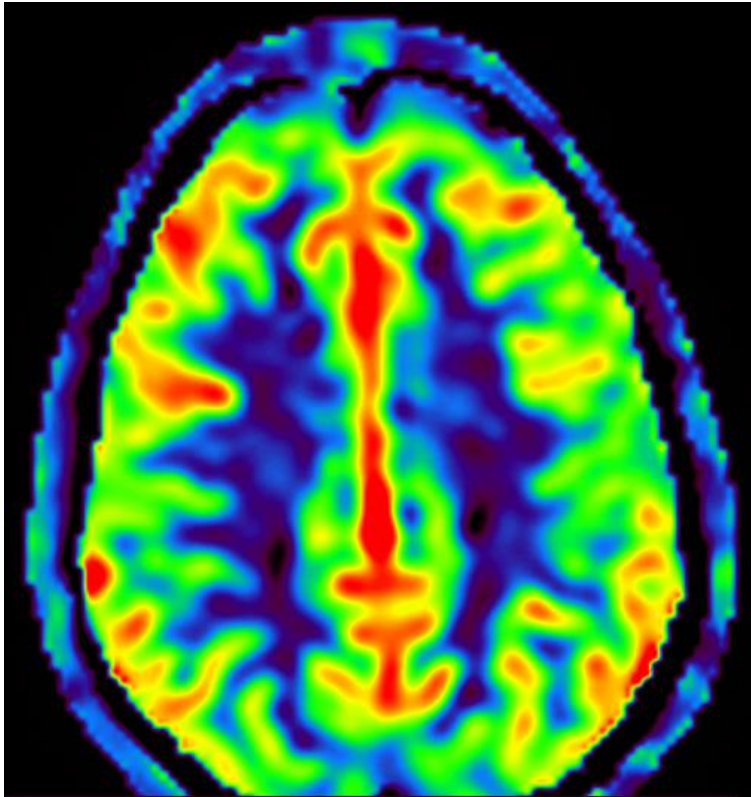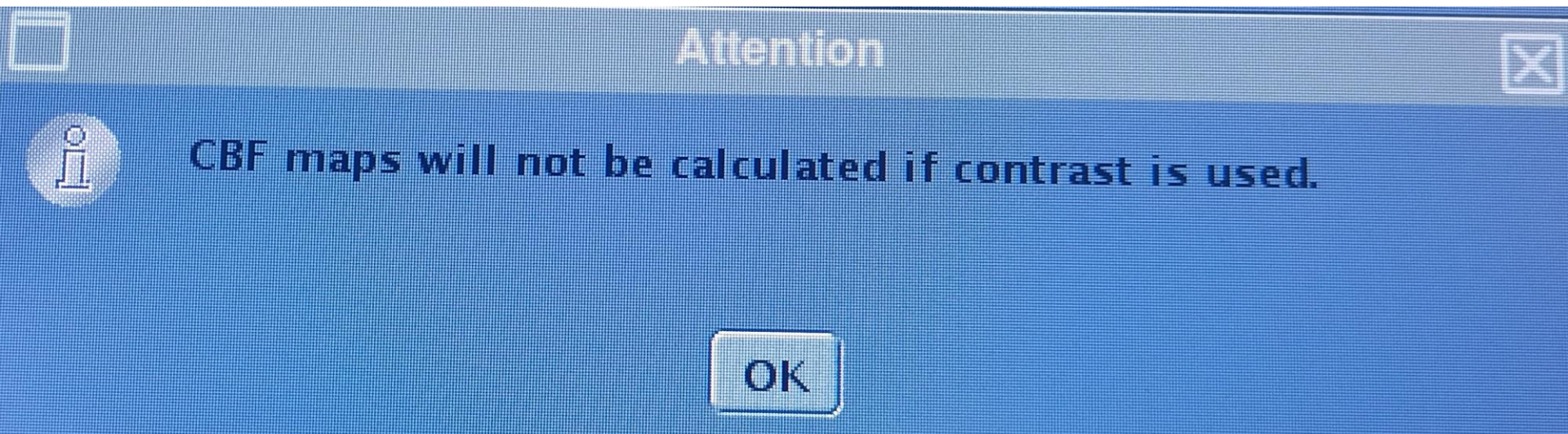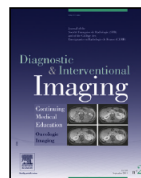ASL Post-MdC

CONTINUING EDUCATION PROGRAM: FOCUS...

Arterial spin labeling (ASL) perfusion: Techniques and clinical use

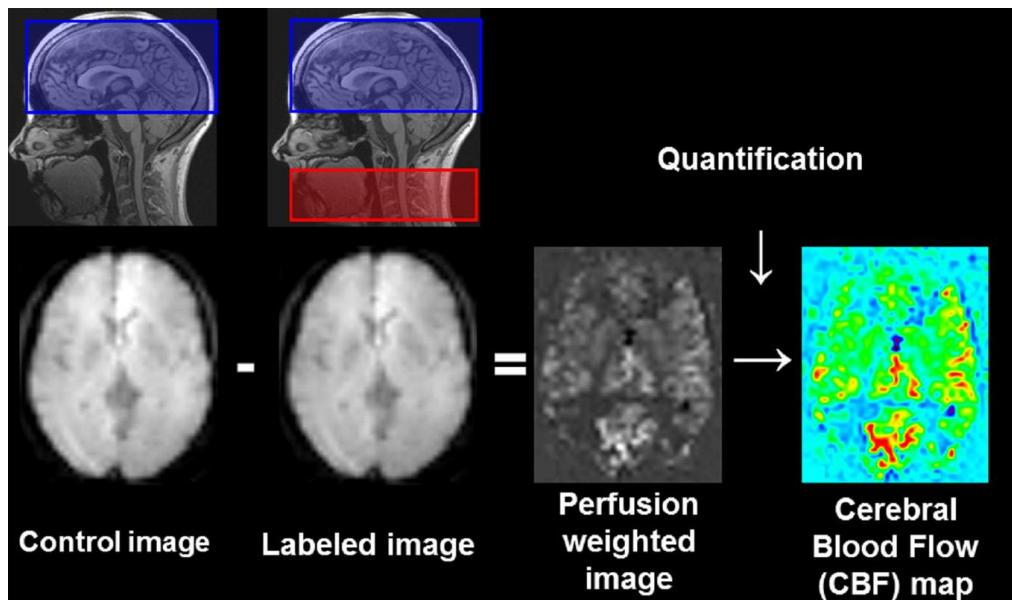
Advantages of ASL

- No irradiation
- No injection of exogenous contrast medium
- Absolute quantification possible of the tissue blood flow and the arterial transit time
- Repeated measurements possible
- Reproducibility

Disadvantages of ASL

- Low signal-noise ratio
- Minimum acquisition time: 3 min
- Limited spatial resolution
- Quantification of the tissue blood volume impossible (ex: cerebral blood volume)
- Absence of harmonization of techniques (type of labeling, image acquisition, post-treatment) between the manufacturers
- Post-treatment optimization, automation and standardization still in development

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